



## EZ Wounds Wound Simulation Set Basic, 4 wounds and consumables

**Order number:** AU-08240

With this casualty simulation set you create extremely realistic wounds for first aid training in a few minutes without having any make-up skills. The molds included in this set can be filled with wound simulation compound many times. The filled molds are pressed onto the required wound position after the compound has dried, the wound adheres to the skin or the manikin without the need of glue. Add some of the supplied artificial blood and the wound is ready for use. For those who require extreme realism a flesh compound can be added into the wound to make it even more realistic. Wounds can be dressed several times, remove the dressing from the wound, wash away the blood and re-apply new blood. Ready for another turn. After finishing the training, the wounds can be easily removed and disposed. No need for cleaning and maintenance.

supplied with:

cut

superficial

deeper laceration

abrasion

wound compound

flesh compound

fixation powder

isopropanol

artificial blood

spatula for applying the compounds and some cotton swabs

for applying the blood

transport box